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- Horien M.P.S. w buteleczce 120ml, 360ml.

ENMULTIPURPOSE SOLUTION

Horien is a multipurpose/disinfecting solution that cleans, rinses, disinfects, stores, removes protein and lubricates all soft contact lenses.

Composition:

Multi/purpose solution is a sterile, buffered, isotonic aqueous solution containing a boric buffer and sodium chloride, with edetate disodium (0.05%) and polzhexamethylene biguanide(0.0001%) as preservatives.

Usage

- Always wash your hand and dry them with a clean towel prior handling your lenses.
- Always start with the same eye to avoid confusion.
- Open the lens case.
- Place the lens in the palm of your hand.
- Apply 3-4 drops of solution to each side of the lens and gently rub with your finger for about 10 seconds.
- Place your lens in between your thumb and forefinger and pour solution.
- Place the lens in the lens case fill one side to the line with fresh solution.
- Repeat same steps for your other lens and close the top.
- For complete disinfection, let your lenses soak for at least 5 minutes.
- To disinfect without rubbing and rinsing, allow your lenses soak for at least 6 hours.
- To improve the comfort, wash your lenses everytime before you use them.
- If the lenses are not used for a long period, freshen your solution every 7 days.
- Discard solution from lens case and rinse it with solution. Leave case open to air dry.

Precautions

- Never touch the dropper tip of the bottle to any surface, since this may contaminate the solution.
- Close bottle after each use.
- Once bottle is open, discard any remaining solution after 2 months.
- Keep out of reach of children.
- Store 15~30°C.
- Store out of direct sunlight.
- Never mix with similar products.
- In Case Of Any Problem, Consult Your EyeCare Professional.

NO THIOMERSAL

Warnings

- Always use fresh solution, never reuse solution in your lens case for a second time.
- Never use solutions passed the expiry date.
- Never use tap water or any other non-sterile water with contact lenses as microbial contamination can occur, which may lead to permanent eye damage.

PACKAGE QUANTITIES

Horien M.P.S. is available in 120ml, 360ml, polypropylene bottles.